



Epilepsy Solutions is a trading name of Independent Living Supplies Ltd. Independent Living Supplies has been established since August 2011, the company's aim was to provide Assistive Technology and Telecare products for those people, young and old who wished to remain independent in their homes or gain more independence through the aid of technology.

As Independent Living Supplies demand for epilepsy monitoring grew and in order to give the correct advice on monitoring equipment we gained more knowledge about the condition by completing Epilepsy Awareness Training.

We have visited hundreds of individuals in their homes and care facilities and gained a real insight into the sometimes devastating effects of people living with epilepsy on a daily basis.

Epilepsy Solutions aim is to provide the right equipment for the monitoring of the condition. Our knowledgeable staff will provide you with the best product advice, best value solutions and a responsive and personalised service.

 **Epilepsy**Solutions **SmartMonitor**
Well Connected Well Protected


Comprehensive Remote Monitoring Solutions



Epilepsy Solutions

Inspyre™

- The leading commercially available and clinically validated wearable device solution designed to detect repetitive shaking motion, similar to those caused by convulsive seizures

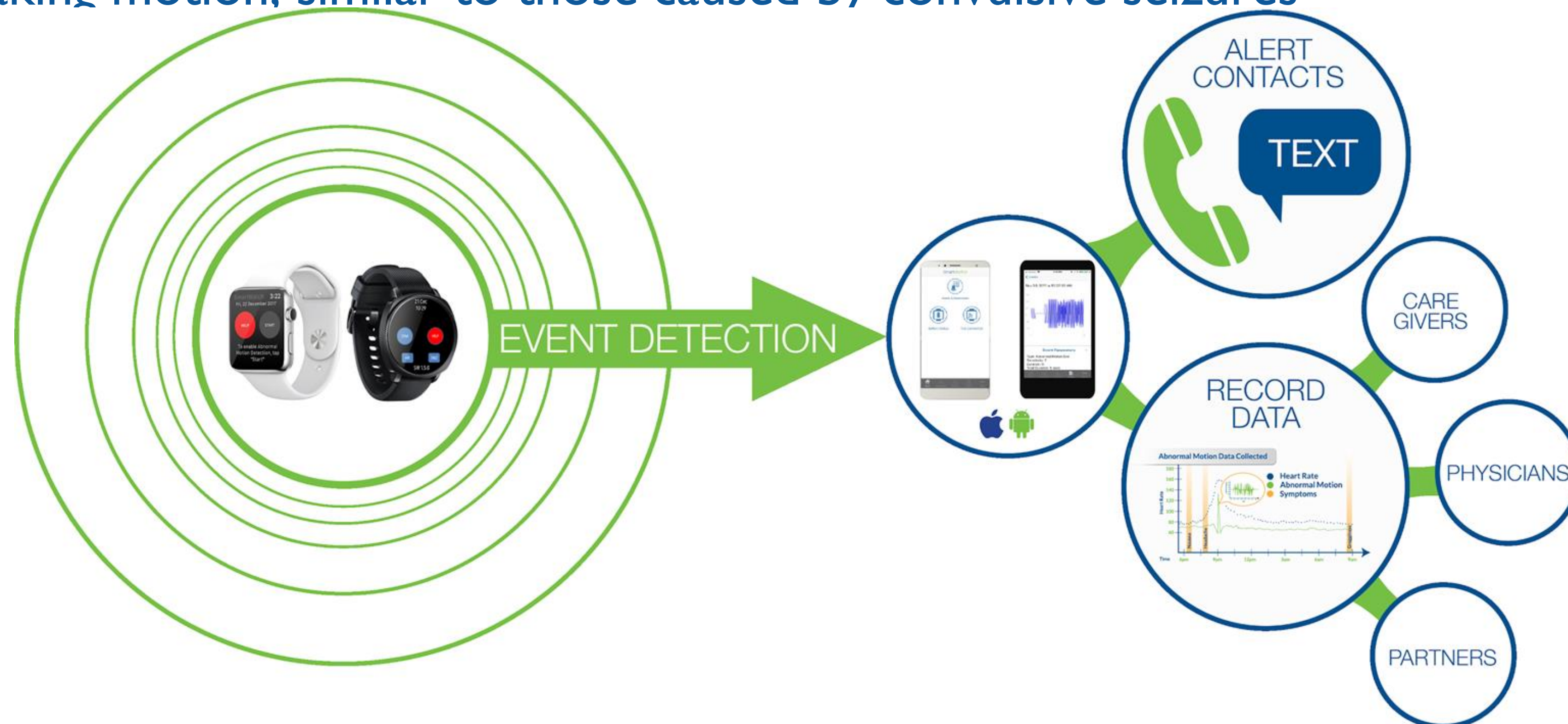

What is the Inspyre™

How does the Inspyre work?

When the Inspyre detects abnormal shaking motion, it instantly sends alerts (audible alarm, texts, and phone calls) to the contacts that the Inspyre user designates, such as multiple family members and care providers. These contacts can provide immediate assistance to the user.



Compatible Devices

Compatible Devices:

Android Users:



Samsung wearables:

- Galaxy Watch
- Galaxy Active 1 or 2
- Galaxy Watch3

iPhone Users:


Apple Watch:

- Series 3 or higher

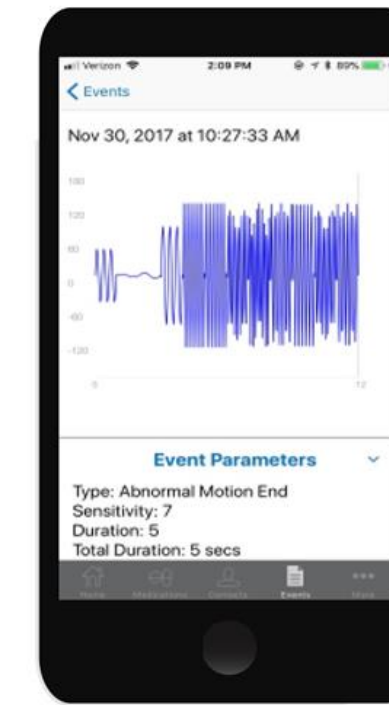
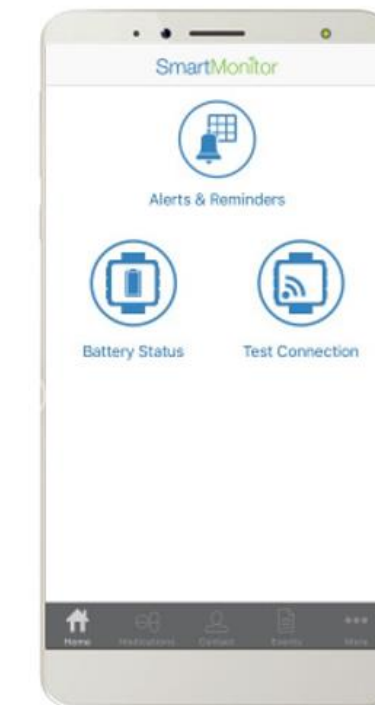


Alerting upon Abnormal Motion

Abnormal Motion Events:



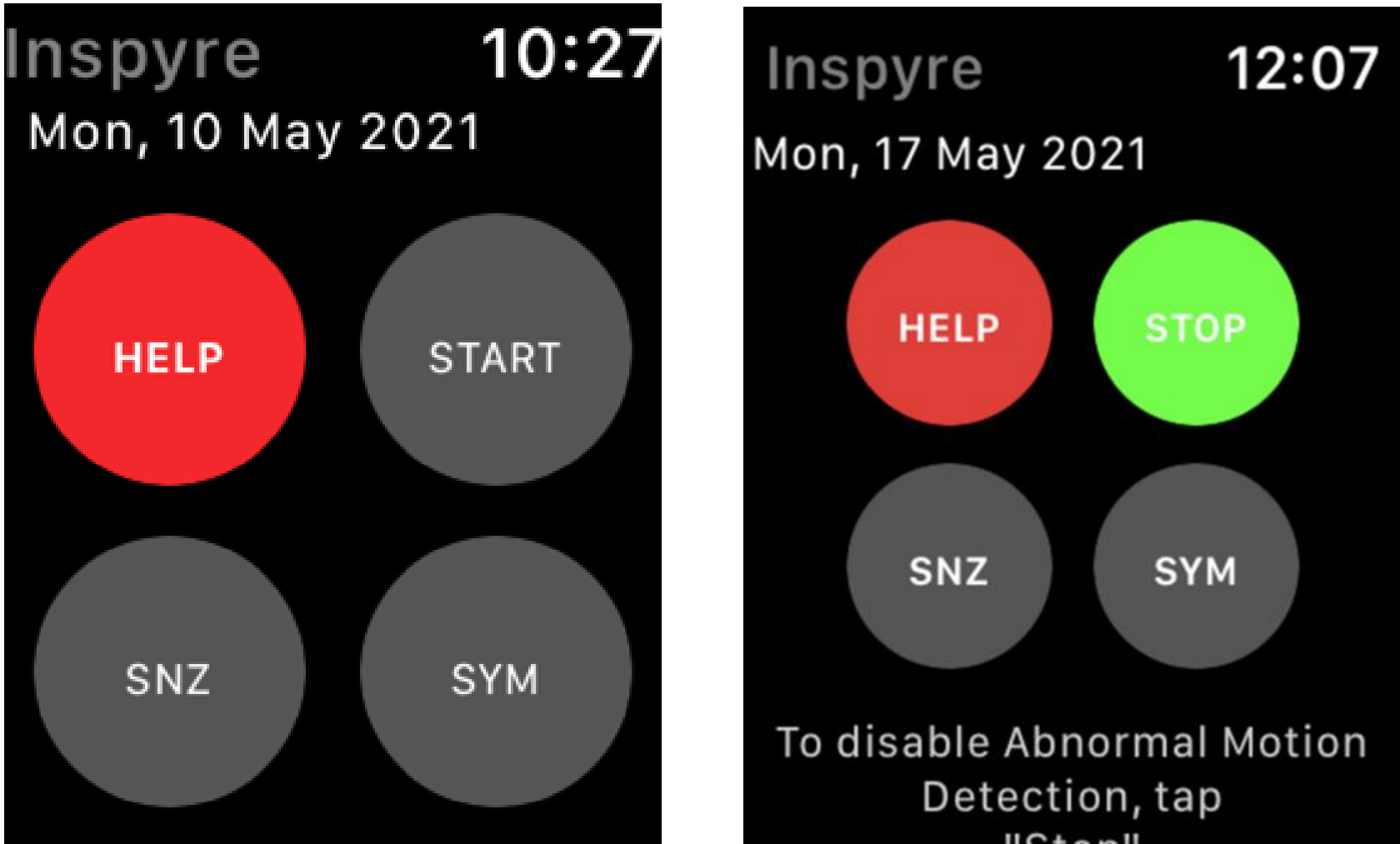


When the Inspyre app on the watch detects abnormal movements, it vibrates on the user's wrist and a cancel button appears. Audible alarms, text, and phone call alerts are automatically sent to the contacts that the user has chosen.

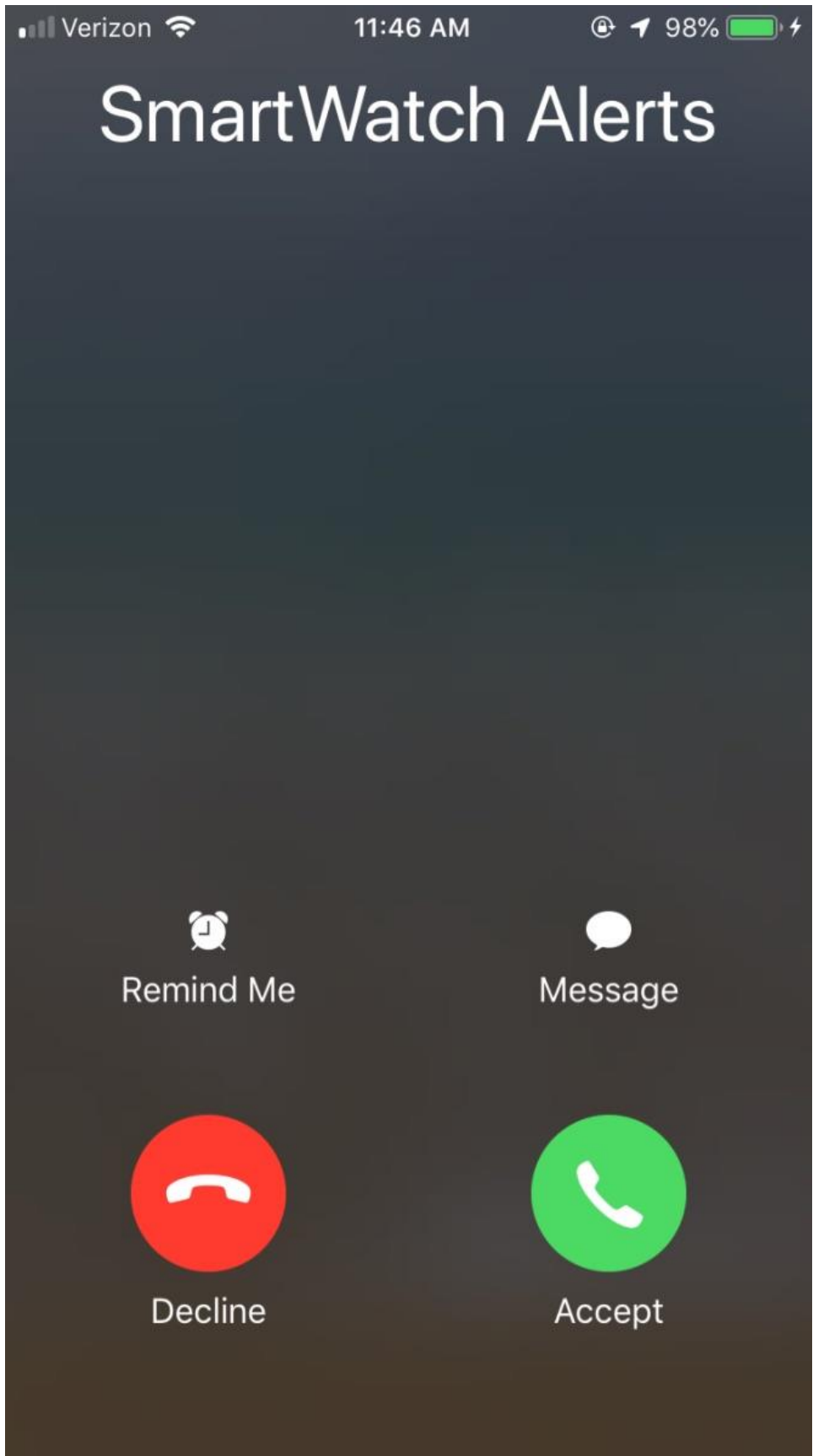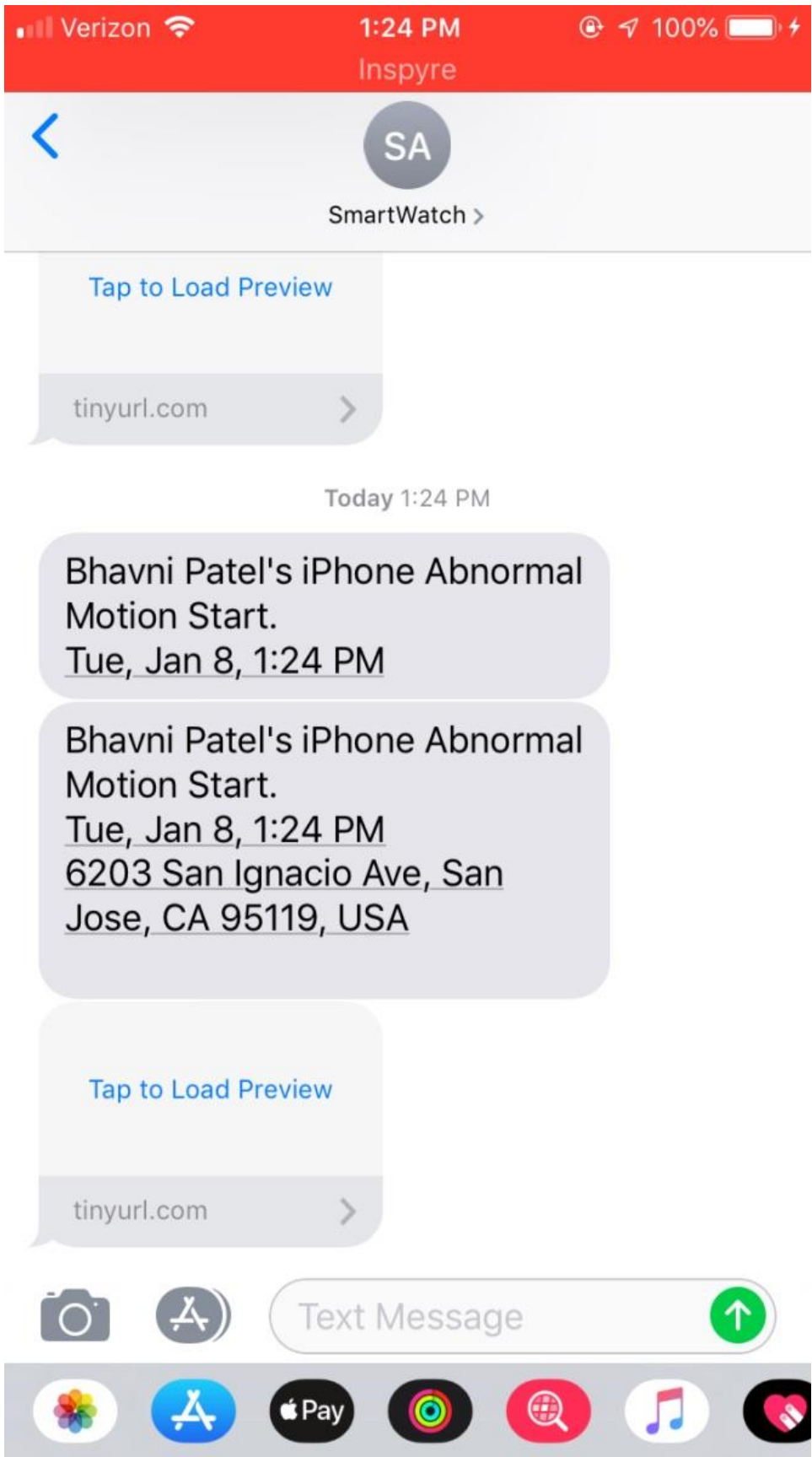


Data Analysis and Reporting

Abnormal Motion Events:

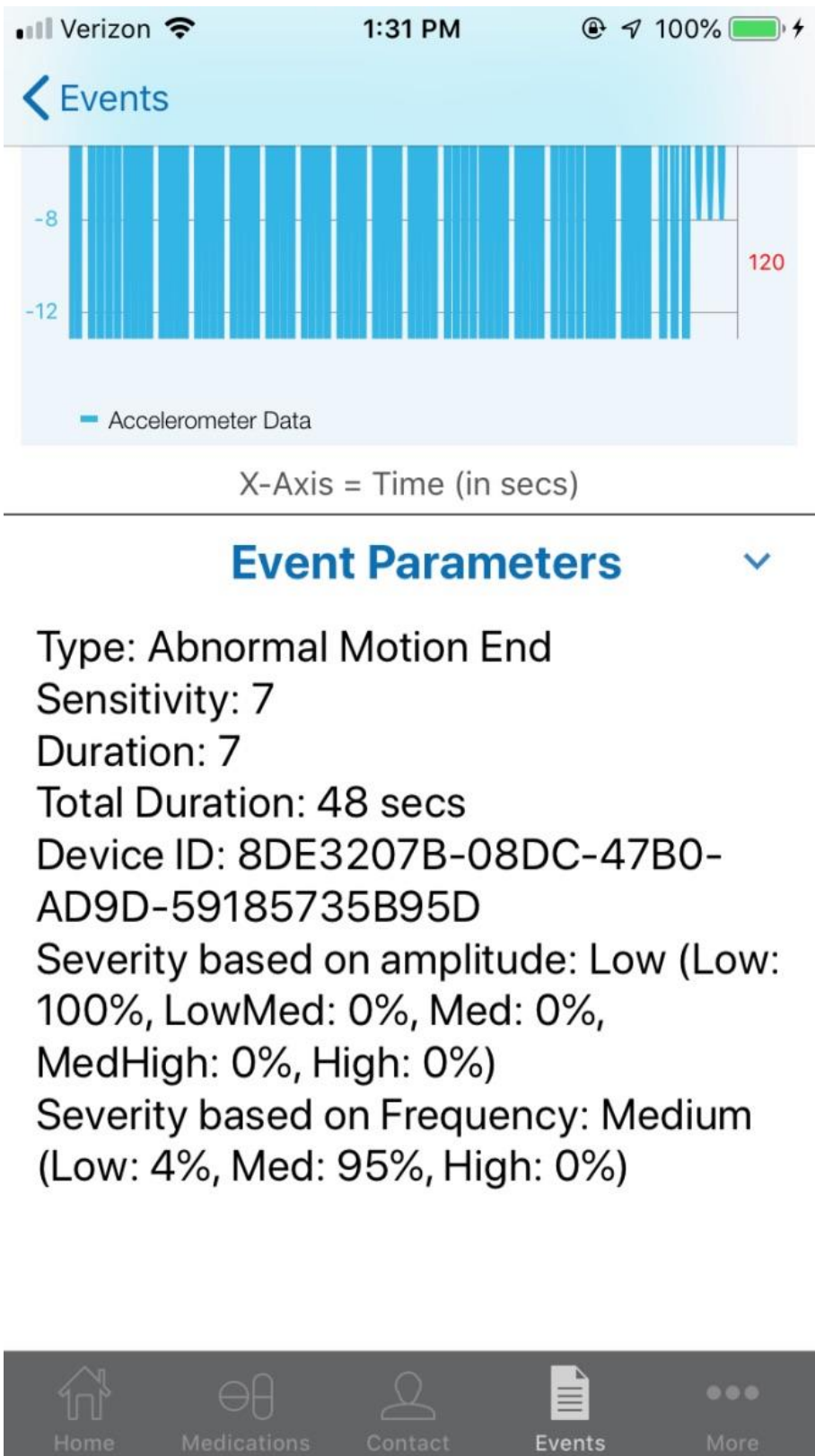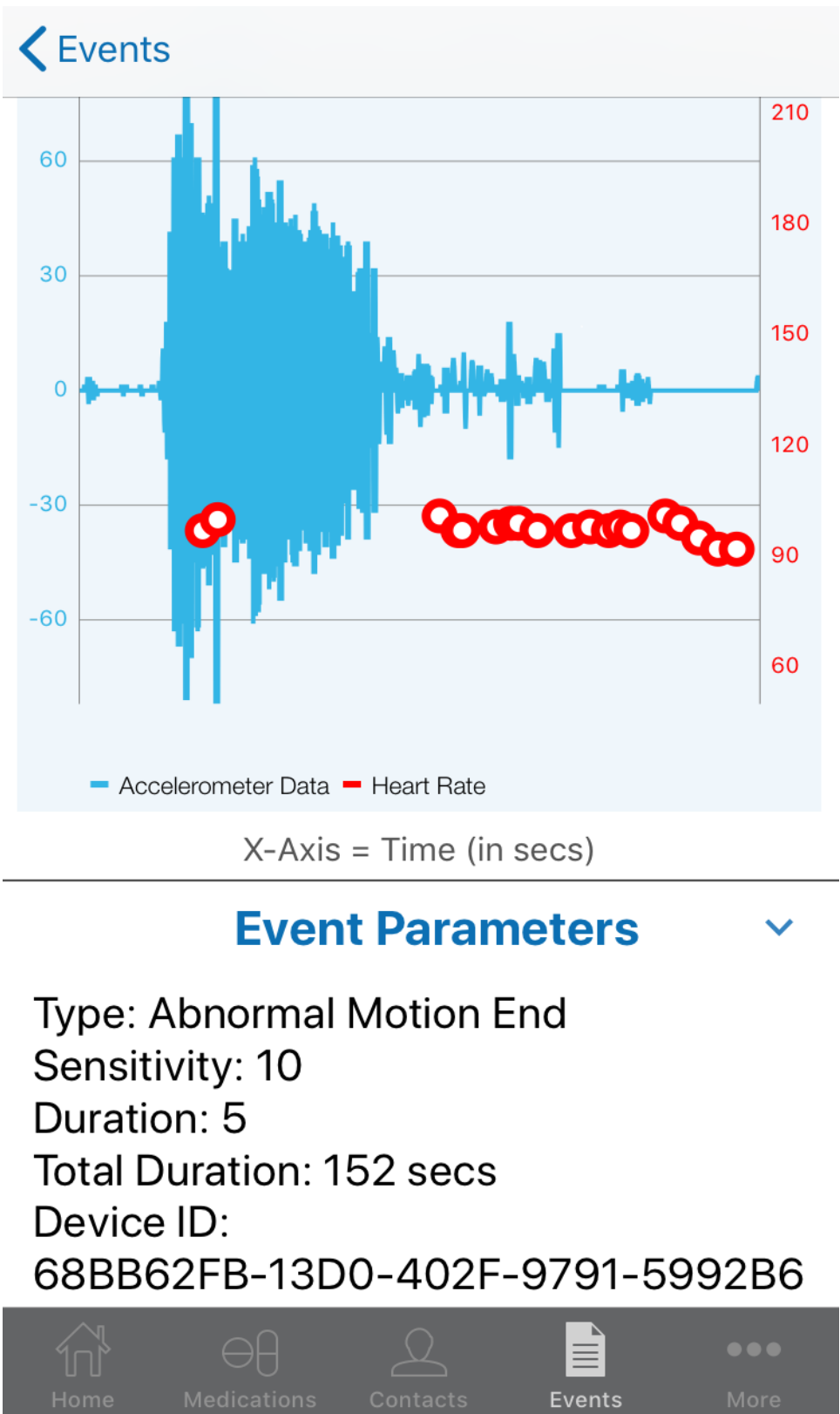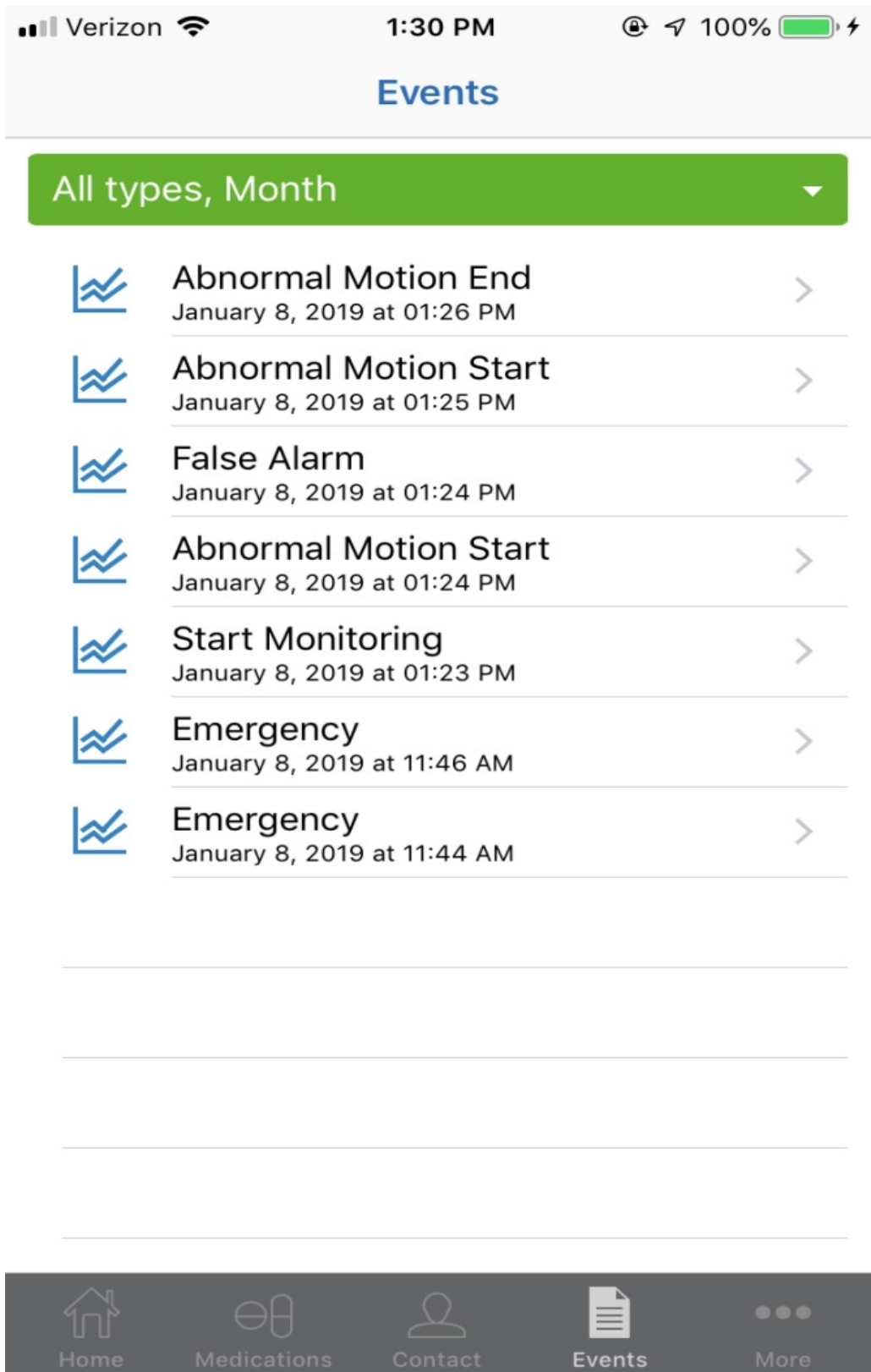

Data collected during the event is stored for later review, both on the phone app and on the user portal.

Unique Features of the Inspyre

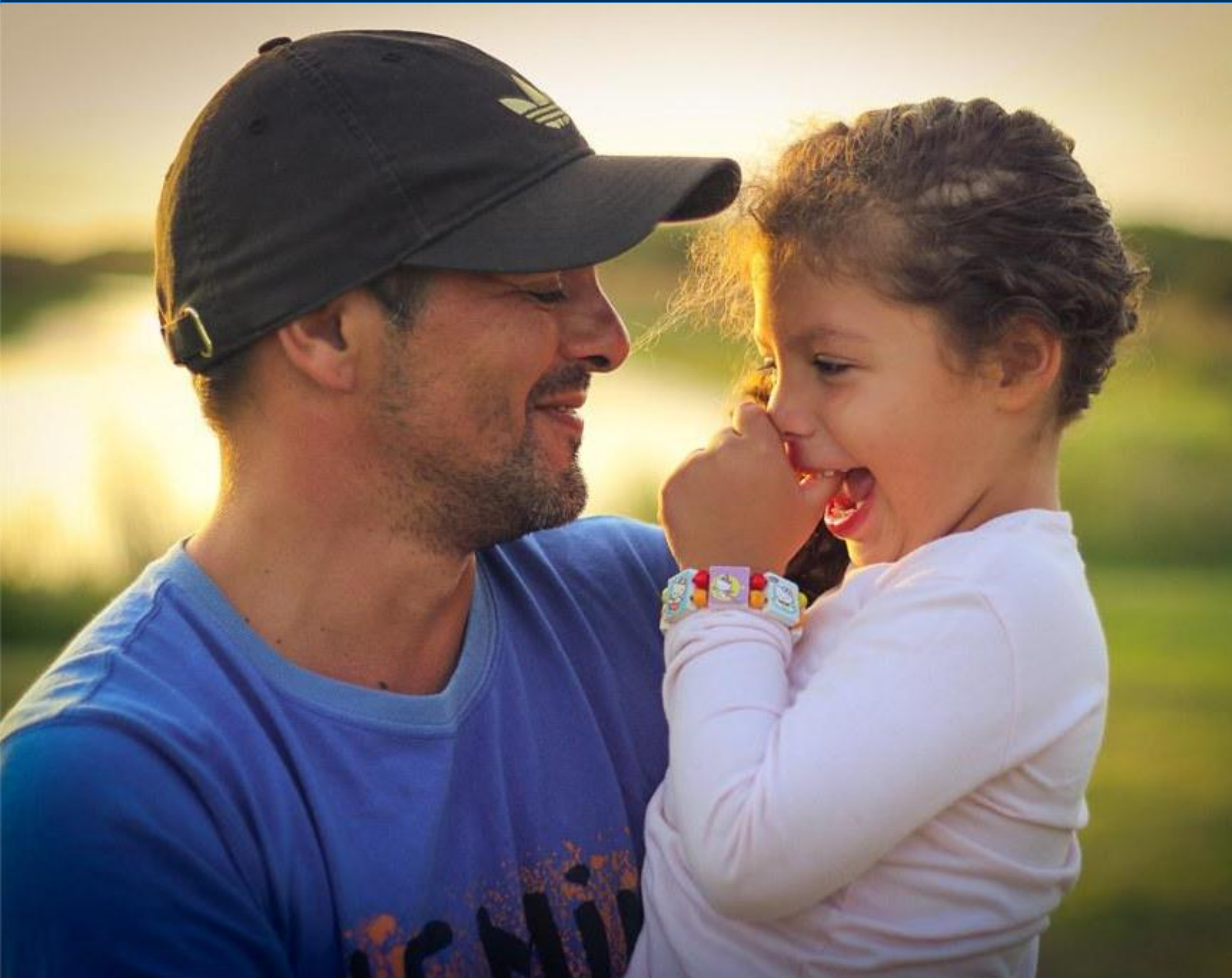

Beneficial Features Include:

- Reminders
- Panic/Help Button
- Adjustable Settings
- Symptom Reporting


The Inspyre™

Reminders:

Users, and their care team, can program reminders into the Inspyre. These reminders can help the user adhere to their medication schedule, when an appointment is underway, to drink enough fluids or eat at certain times, and more.

The reminders will show on the user's watch and phone, and the reminder can also be sent to any contacts selected.



The screenshot shows the 'Medications' screen in the Inspyre app. It features a back arrow and the title 'Medications'. Below this is a text input field for 'Enter Medication Name'. A section titled 'Set Form & Quantity' contains two green dropdown menus for 'Form' and 'Quantity'. Below that is a 'Dosing Instructions' section with a text input field. The 'Dose Reminder(s)' section includes five green dropdown menus for 'Type', 'Take', 'Frequency', 'Schedule from', and 'Start date' (set to '01/09/2019'). At the bottom, there is a navigation bar with icons for Home, Medications, Contacts, Events, and More.

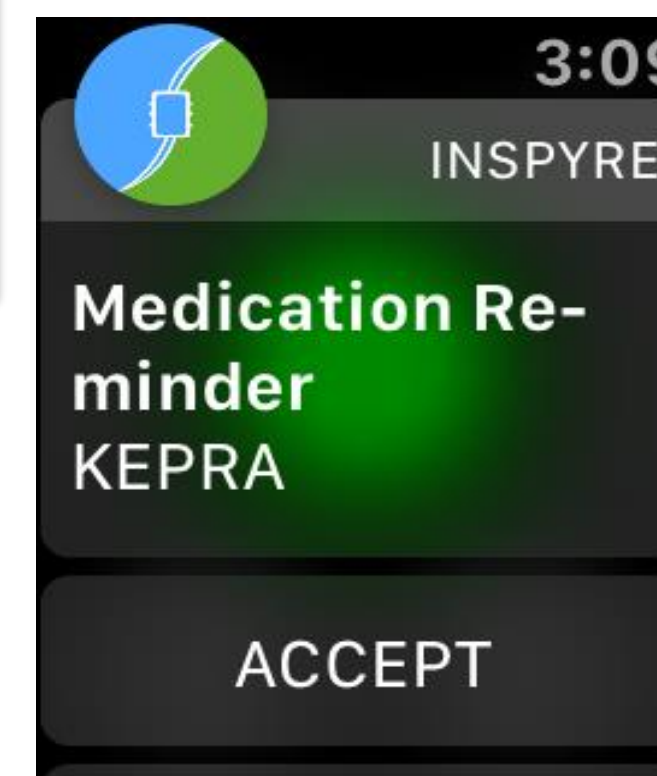

Panic/Help button

Panic/Help button for timely assistance:

A **panic/help button** to alert contacts in real-time of an emergency/urgent situation.

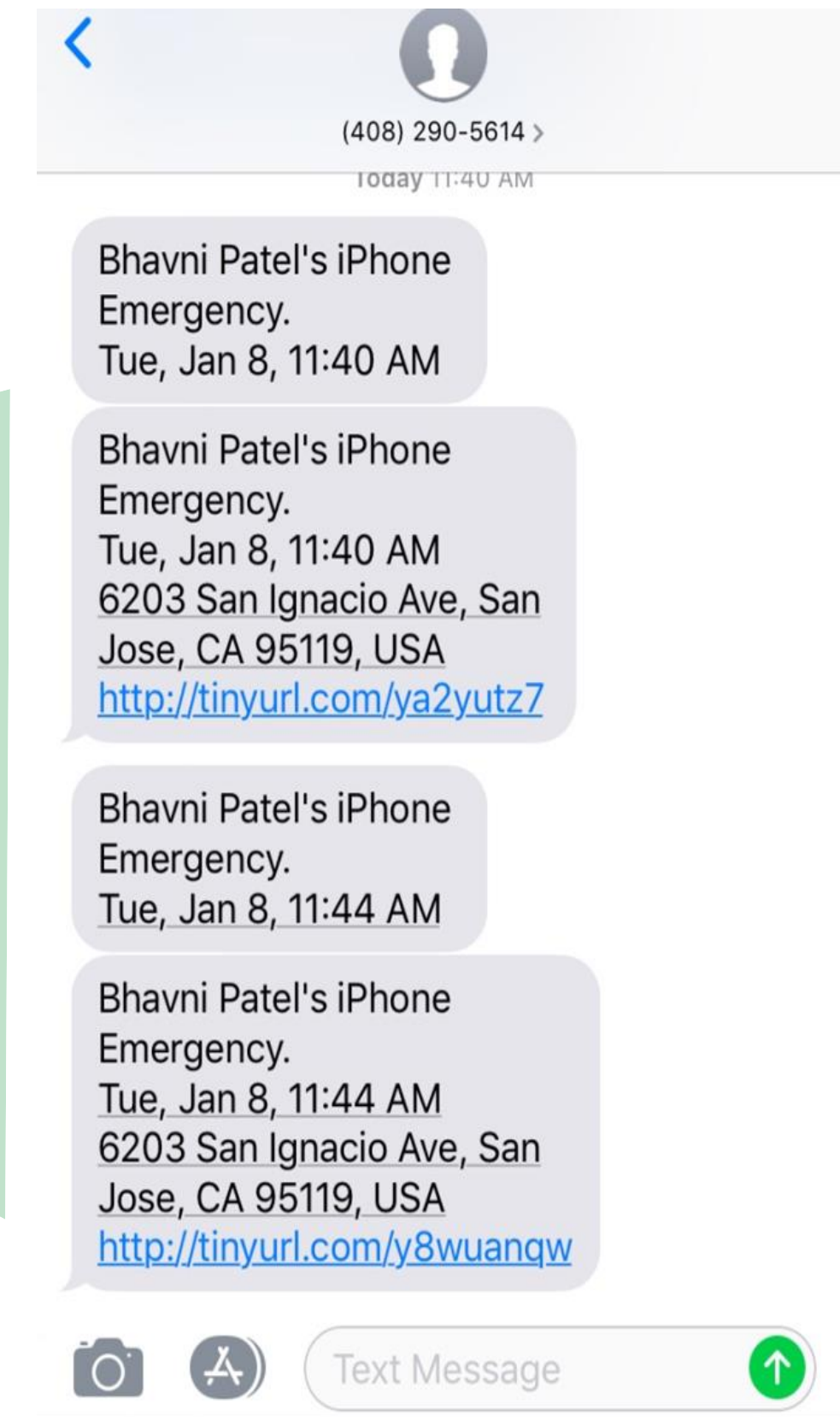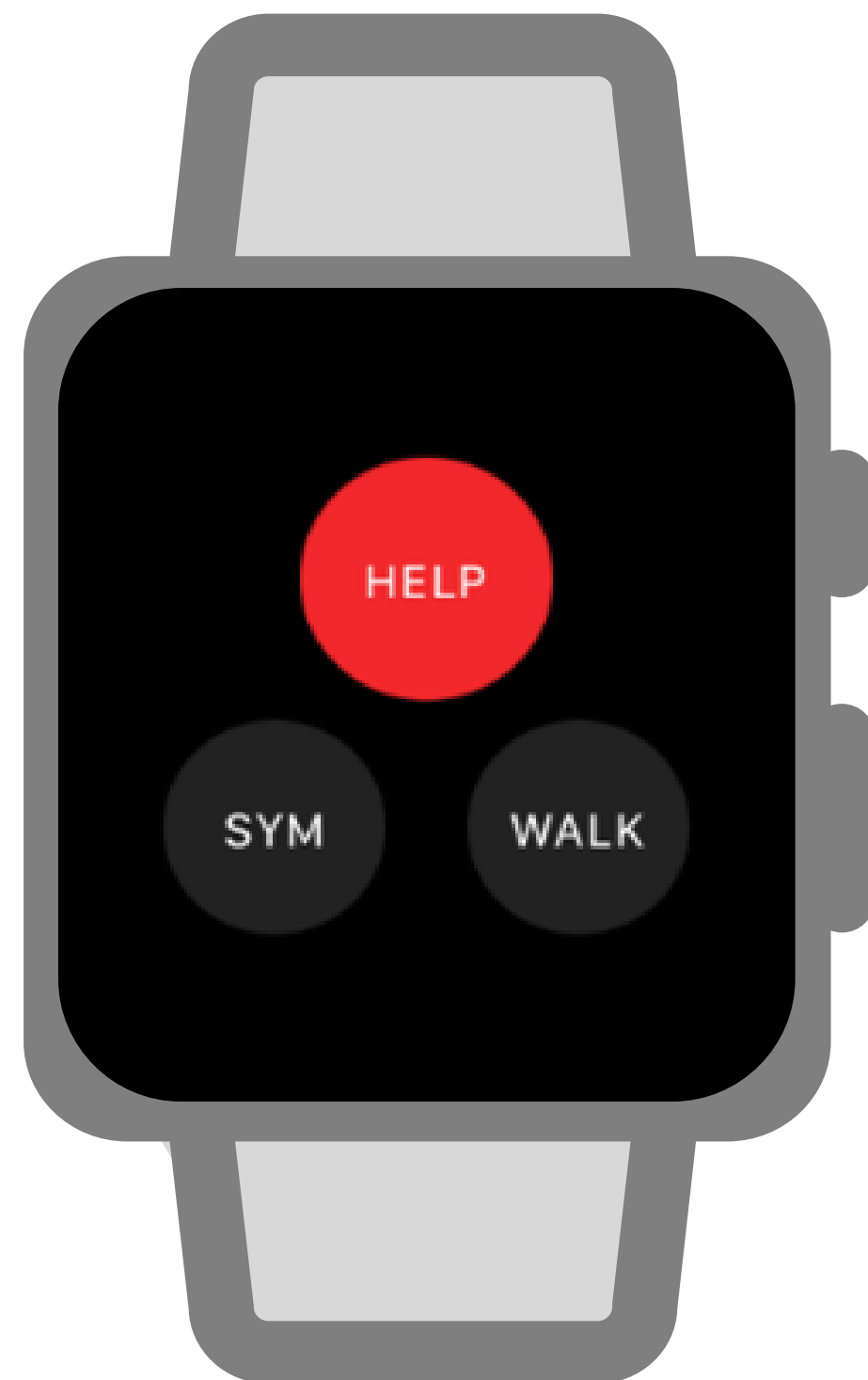

Adjustable Settings

Adjustable Settings:

The Inspyre comes with the option to **adjust daytime and nighttime sensitivity settings** to customize it to each user's needs and helps minimize false alarms.

Settings

More

MY PROFILE

FEATURES

INSPYRE

Daytime Settings

06:30 AM

TO

08:30 PM

Sensitivity (1-10):

10

Duration (4-20 seconds):

5

Nighttime Settings

08:30 PM

TO

06:30 AM

Sensitivity (1-10):

10

Home

Medications

Contacts

Events

More






Reporting Symptoms

Symptom Reporting:

The ability to **report symptoms** from the watch and/or phone app.


Symptom reports are available for later review.

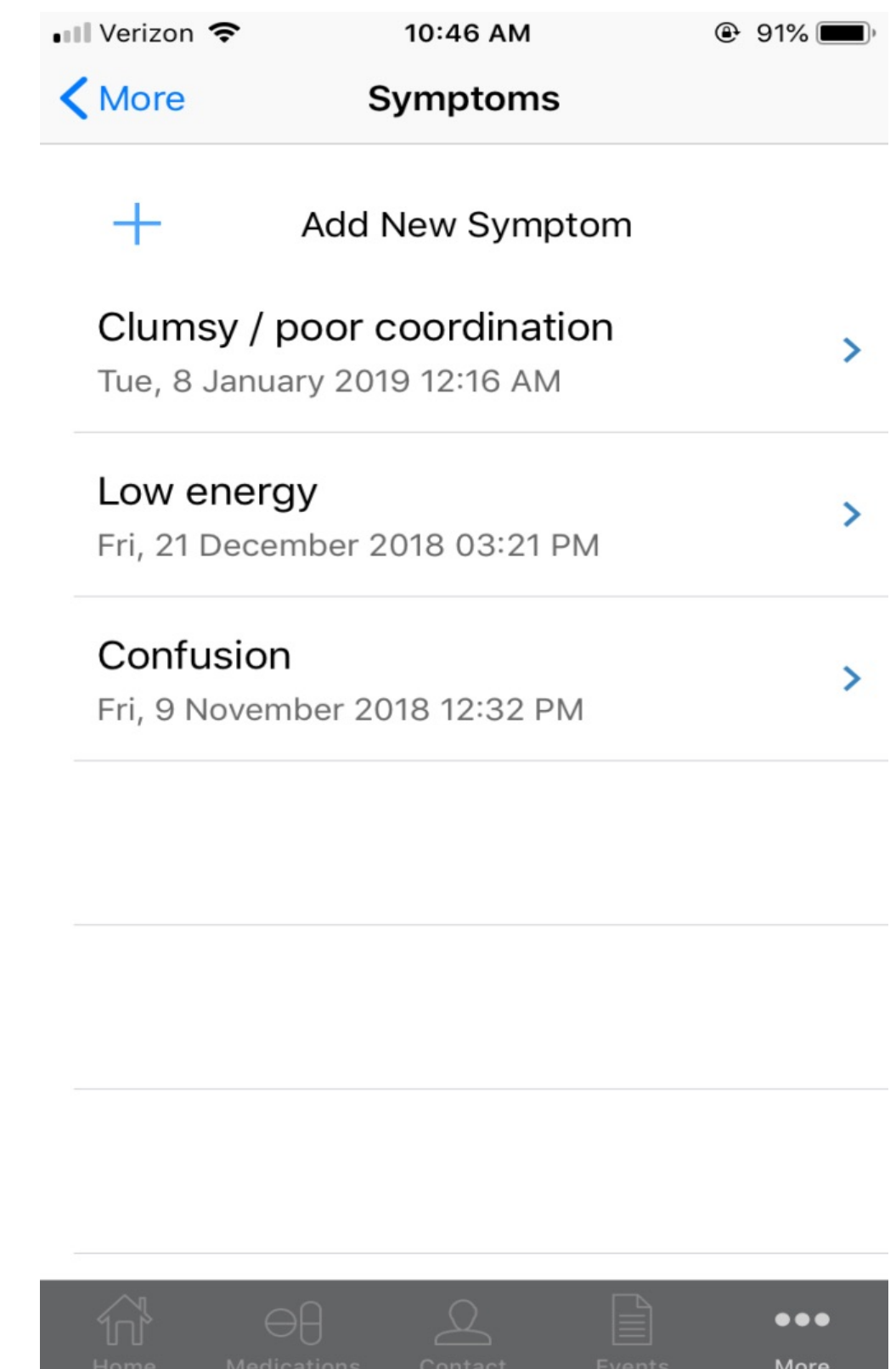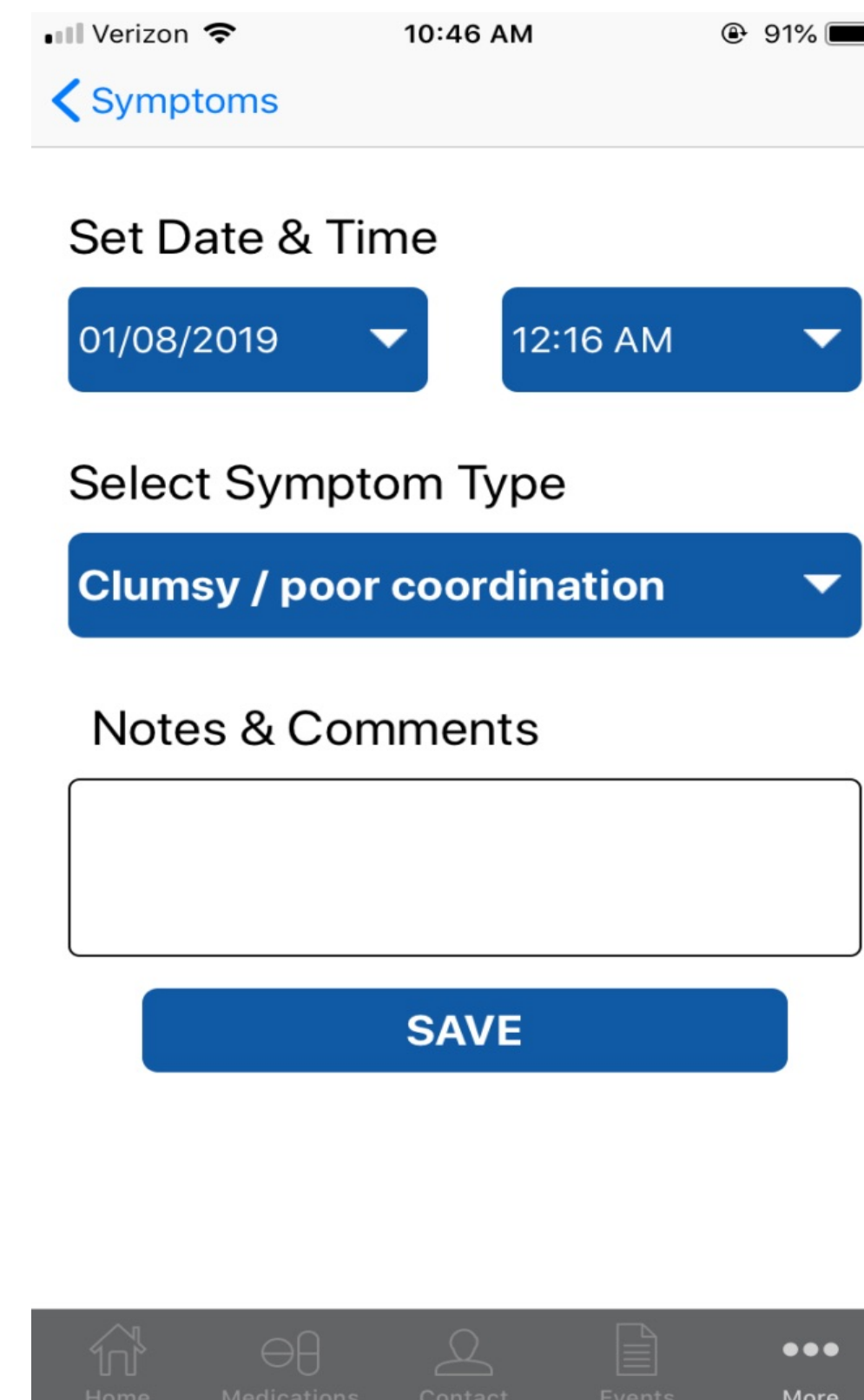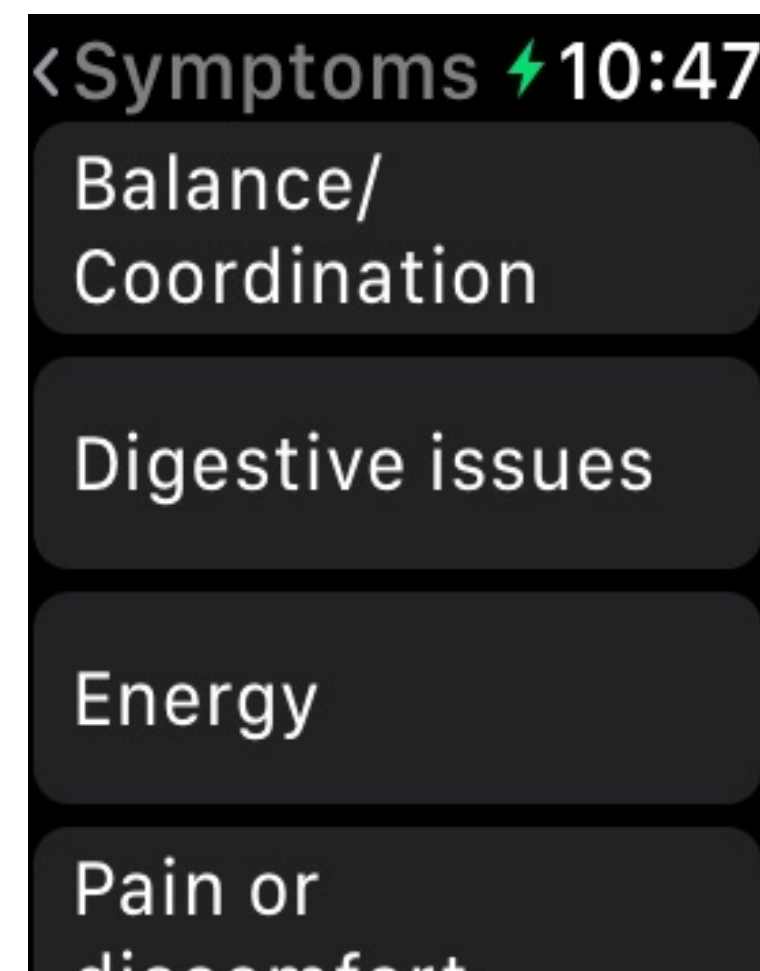

Real-time dashboards and Reporting

User Dashboard:

Inspyre™ records multiple data points pertaining to the episode, including the time, date, location, duration, and severity of all alerted events, as well as heart rate and symptoms experienced. Data is available for secure access and review by users and their care teams.

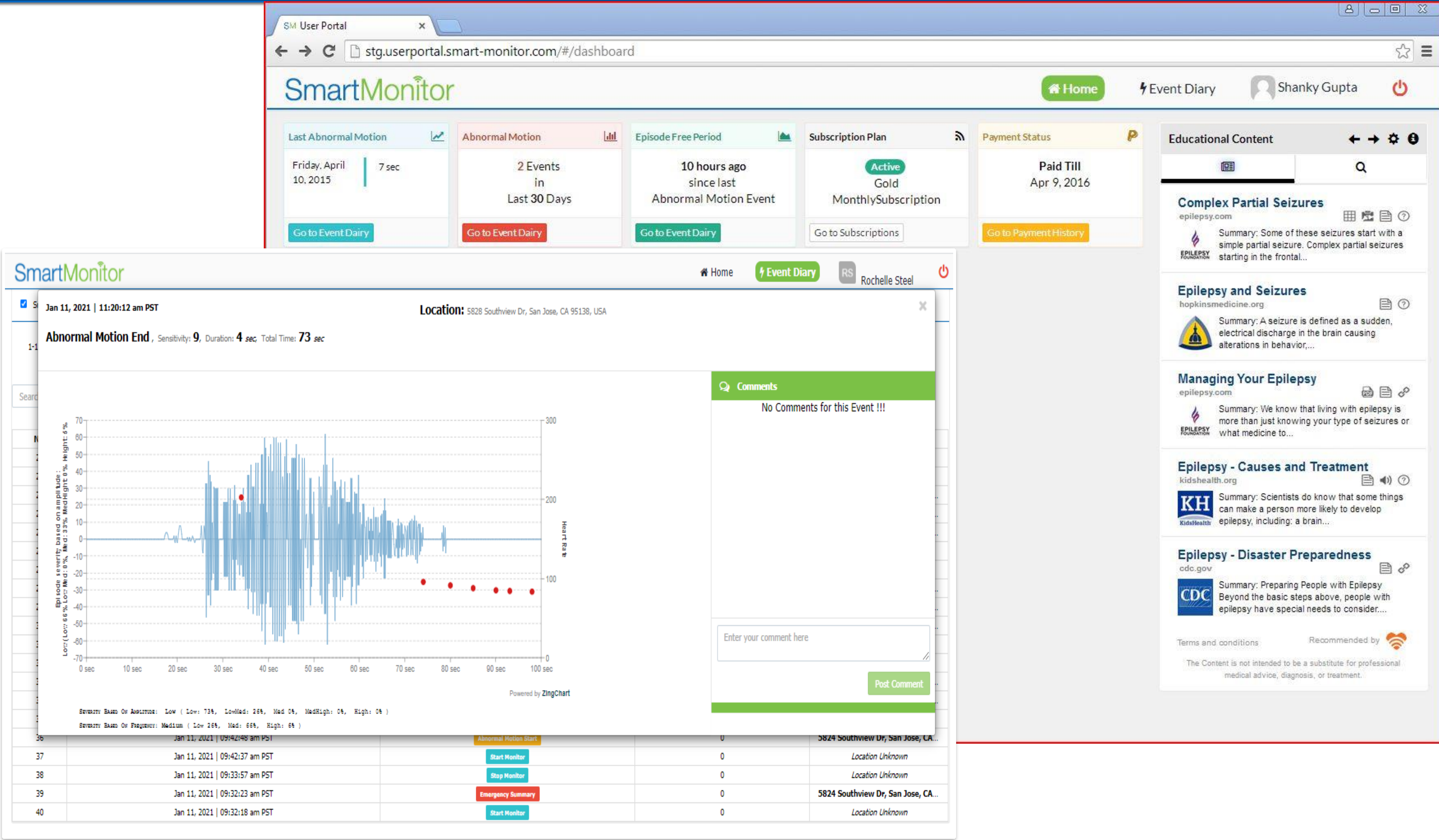

EpilepsySolutions

Inspyre™ by Smart Monitor Features and Pricing - Android (does not include Samsung watch)				Inspyre™ by Smart Monitor Features and Pricing - iOS (does not include Apple Watch)			
Feature	Bronze	Silver	Gold	Feature	Bronze	Silver	Gold
Abnormal Motion Detection	✓	✓	✓	Abnormal Motion Detection	✓	✓	✓
Cancel Button	✓	✓	✓	Cancel Button	✓	✓	✓
Help/Emergency Button		✓	✓	Help/Emergency Button		✓	✓
Snooze (pause) Button		✓	✓	Snooze (pause) Button		✓	✓
Audio Recording During Event			✓	Audio Recording During Event			✓
Heart Rate Data Collected			✓	Heart Rate Data Collected			✓
GPS Location			✓	GPS Location			✓
Instant Text Alerts (number of contacts) (Two contacts receive phone calls)	1 contact	5 contacts	Unlimited	Instant Text Alerts (number of contacts) (Two contacts receive phone calls)	1 contact	5 contacts	Unlimited
Medication Reminders			✓	Medication Reminders			✓
Symptom Reporting			✓	Symptom Reporting			✓
Monthly Pricing	£9.99	£19.99	£24.99	Monthly Pricing	£14.99	£24.99	£29.99
Activation	£19.99	£19.99	£19.99	Activation	£19.99	£19.99	£19.99

Contact

Lisa Cox

Director

lisa@independentlivingsupplies.co.uk

Lorina Felicien

Development Manager

info@epilepsysolutions.co.uk

Epilepsy Solutions

www.epilepsysolutions.co.uk